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**HK Pregnancy Tester
for Pigs
Operating Instructions**

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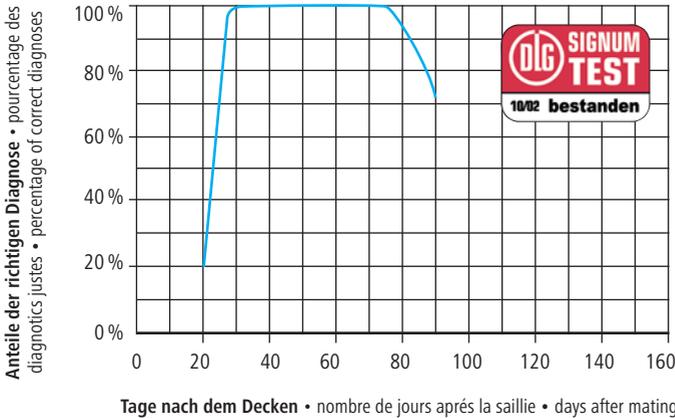
The probe should be placed approx. 5-6 cm above the teatline, before the second teat from behind and directed in an angle of approx. 45° upwards against the animal's body.

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Direction of the probe and uterus' position of a nonpregnant (1) and a 4-5 week pregnant sow (2)

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Results of a pregnancy diagnostics test from 362 sows between the 16th and 75th day after mating.

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Warranty

Albert Kerbl GmbH grants you an 2-year guarantee on this article in accordance with our general guarantee conditions. These general warranty conditions can be found at www.kerbl.com/guarantee. We will also be happy to send you the warranty conditions by post. Your statutory warranty rights remain unaffected by our guarantee.

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CE-/UKCA-conformity declaration

Albert KERBL GmbH hereby declares that the product / device described in these instructions complies with the fundamental requirements and other relevant stipulations and regulations. The CE-/UKCA-mark confirms compliance with the Directives of the European Union or the relevant UK legislation.

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Electronic scrap

Disposing of this device after its service life is the responsibility of the operator. Please consult the valid national regulations. The device must not be disposed of in household waste. In accordance with the stipulations of the EU Directive on the Disposal of Electrical and Electronic Devices, the device can be disposed of free of charge at the local waste collection or recycling centre. Alternatively, it can be returned to retailers who offer a collection service. The proper disposal helps to ensure environmental protection and prevents any adverse effects on human health and the environment.

Albert Kerbl GmbH

Felizenzell 9 84428 Buchbach, Germany www.rheintechnik.de
info@kerbl.com +49 8086 933-100

Kerbl UK Ltd

Lands End Way Oakham, Rutland LE15 6RF, UK
www.kerbl.co.uk